



Giardia-Strip


Rapid diagnostic test for *in vitro* detection of *Giardia* in stool specimens

- Easy to use
- Easy to read
- Specific
- Sensitive
- Reliable
- Rapid (15')



www.corisbio.com

Giardia-Strip

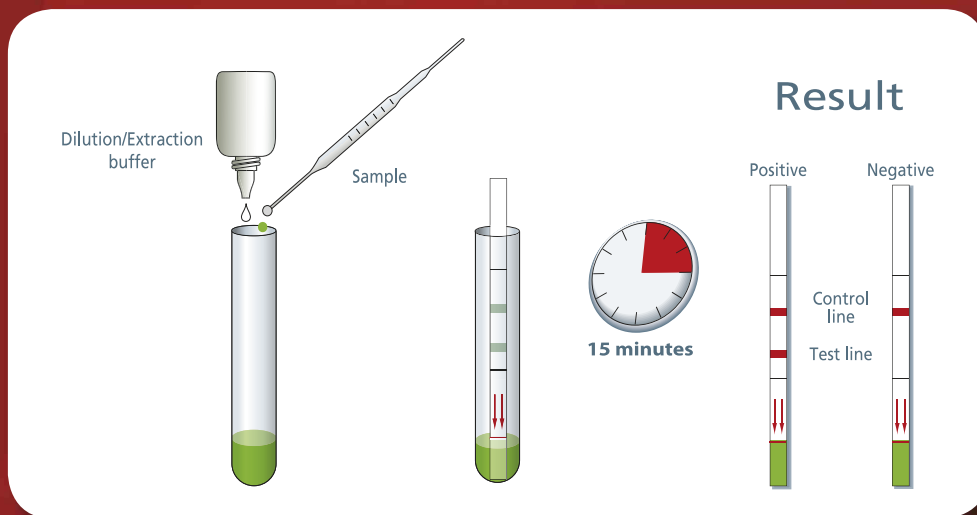

Performance compared to

| | ICT competitor (n=100) Spain (Saragosse) | Microscopy (n=382) France (Lyon) |
|-------------|---|-------------------------------------|
| Specificity | 100% (52/52) | 97.8% (309/316) |
| Sensitivity | 89.6% (43/48) | 96.3% (77/80) |
| PPV | 100% (43/43) | 91.7% (77/84) |
| NPV | 91.2% (52/57) | 99% (309/312) |
| Accuracy | 95% (95/100) | 97.5% (386/396) |

For more technical information

client.care@corisbio.com


Ordering Information

| Product Name | Description | Storage | Code |
|---------------------------------------|---|-------------|--------|
| Giardia-Strip | 25 tests, buffer | 4°C to 30°C | C-1013 |
| Giardia K-SeT | 20 cassettes, buffer | 4°C to 30°C | K-1213 |
| Giardia K-SeT, with collection set | 20 cassettes, 20 faecal sampling systems | 4°C to 30°C | K-1513 |

All diagnostics available in Coris BioConcept catalogue :

Adenovirus, Adenovirus (Respiratory), Adenovirus 40 and 41, *Clostridium difficile*, Bovine Coronavirus, *Cryptosporidium*, *Escherichia coli* O157, *Escherichia coli* K99, Giardia, *Helicobacter pylori*, Influenza A and B viruses, *Legionella pneumophila*, Respiratory Syncytial Virus, Rotavirus, Mefloquine, Proguanil, *Leishmania spp.*, *Trypanosoma cruzi*, *P. aeruginosa* efflux.

Distributed by:

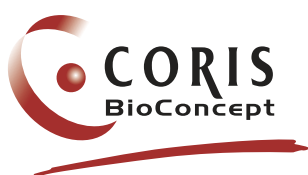

Science Park CREALYS
Rue Jean Sonet 4A - 5032 Gembloux - Belgium
Ph.: +32 (0) 81 719 910 - Fx.: +32 (0)81 719 919
sales@corisbio.com - www.corisbio.com

MADE IN BELGIUM

Fo 7.5-32-13/14
January 2012